

## DROP MECHANISM LUBRICATION

Periodic lubrication of the drop mechanisms will insure smooth, crisp drops. We recommend that you use 3-IN-ONE oil after about 500 adjustments or once/month, whichever comes first. If the drops have not been lubricated on a regular basis, it is recommend to first clean the plungers using WD-40. See instructions below.

### CERVICAL DROP



#### Routine lubrication:

1. Raise the back of the headpiece.
2. Apply 3-IN-ONE into the hole of the drop mechanism housing as shown in the photo. Allow a few minutes for the lubricant to penetrate the mechanism before using the headpiece drop.

#### If the drop has not been lubricated on a regular basis:

1. First clean the plunger from accumulated dust and debris by applying WD-40 (as described above) and then drop the headpiece several times.
2. Apply 3-IN-ONE oil to lubricate the drop (as described above).

## UPHOLSTERY

Normal day to day soil on the vinyl upholstery can be removed with neutral soap and warm water followed by a thorough water rinse. Avoid harsh detergents and powdered abrasives. Areas coming in contact with hair, body oils or perspiration should be washed frequently. Remove stains immediately to prevent the possibility of them becoming permanent. In the event of damage, pre-sewn replacement upholstery is available from Thuli Tables.

### THORACIC & PELVIC DROPS



#### Routine lubrication:

1. Cock the thoracic and pelvic drop sections and then tip the table onto its side with the tension control knobs. This will expose the lubrication hole in each of the drop mechanism housings (on the opposite side of the tension control knobs).
2. Apply 3-IN-ONE oil into this hole. Allow a few minutes for the lubricant to penetrate the mechanisms before using the drops.

#### If the drops have not been lubricated on a regular basis:

1. First clean the plungers from accumulated dust and debris by applying WD-40 (as described above) and then drop each section several times.
2. Apply 3-IN-ONE oil to lubricate the drops (as described above).