

## PELVIC DROP: LUBRICATION

Periodic lubrication of the pelvic drop mechanism will insure a smooth, crisp drop. We recommend that you use 3-IN-ONE oil after about 500 adjustments or once per month, whichever comes first. If the drop has not been lubricated on a regular basis, it is useful to first "clean" the plunger using WD-40. See the following instructions.

### Routine lubrication:

1. Tip the table onto its side. Locate the lubrication hole in the drop mechanism housing (located on the opposite side of the cocking handle).
2. Apply 3-IN-ONE oil into this hole. Allow a few minutes for the oil to penetrate the mechanism before using the drop.

### If the drop has not been lubricated on a regular basis:

1. Tip the table onto its side. Locate the lubrication hole in the drop mechanism housing (located on the opposite side of the cocking handle).
2. First clean the plunger of accumulated dust and debris by applying WD-40 into the lubrication hole. Wait a few minutes for the WD-40 to penetrate the mechanism and then drop the section several times.
3. To lubricate the plunger, apply 3-IN-ONE oil into the lubrication hole. Allow a few minutes for the oil to penetrate the mechanism before using the drop.



## UPHOLSTERY

Normal day to day soil on the vinyl upholstery can be removed with neutral soap and warm water followed by a thorough water rinse. Avoid harsh detergents and powdered abrasives. Areas coming in contact with hair, body oils or perspiration should be washed frequently. Remove stains immediately to prevent the possibility of them becoming permanent. In the event of damage, pre-sewn replacement upholstery is available from Thuli Tables.