

PELVIC DROP: CLEANING & LUBRICATION

Periodic lubrication of the pelvic drop will ensure a smooth, crisp drop. Apply **3-IN-ONE oil** every month or 500 adjustments. If the drop has not been lubricated on a regular basis, first apply **WD-40** to clean the plunger from dust & debris.

Routine lubrication:

1. Place the table on its cushions.
2. Locate the stainless steel plunger of the pelvic drop.
3. Apply **3-IN-ONE oil** onto the plunger. Allow a few minutes for the oil to penetrate before setting the table upright.

If the drop has not been lubricated on a regular basis:

1. Lay the table on its cushions.
2. Apply **WD-40** onto the plunger and allow a few minutes for penetration. Set the table upright and drop the section several times to clean from dust & debris.
2. Set the table back on its cushions and apply **3-IN-ONE oil** onto the plunger for lubrication. Allow a few minutes for the oil to penetrate before setting the table upright. Drop the section several times.



TENSION CONTROL ADJUSTMENT

The resistance on the tension control rod can be increased or decreased. Locate the set screw on the bottom of the pelvic drop block (see photo). Use an 1/8" allen wrench to adjust.

Increase resistance: Turn the set screw clockwise. This is helpful if the tension control rod is "backing out" while using the drop.

Decrease resistance: Turn the set screw counter-clockwise.



Set Screw

VINYL CLEANING

1. **Mild Daily Cleaning:** Use dish soap and warm water (1:10) with a soft cloth. **Follow with a thorough, clear water rinse.** If more cleaning is necessary, use a soft bristle brush with the same solution. Avoid harsh detergents and powdered abrasives. Areas coming in contact with hair, body oils or perspiration should be washed frequently. Remove stains immediately to prevent the possibility of becoming permanent.
2. **Disinfecting Options:**
 - Bleach:** Dilute 4 tsp bleach in 1 quart of water in a spray bottle. Spray on vinyl, leave for 1-2 minutes and rinse with water.
 - Hydrogen Peroxide (3%):** Available in spray bottle. Use undiluted and leave on for 5 minutes and rinse well with water.
 - Alcohol (70% min):** Use undiluted. Leave on for 1-2 minutes and rinse well with water.
3. **Water Rinse:** It is important to follow cleaning with a thorough, clear water rinse to minimize premature deterioration of the vinyl from extended exposure to chemicals.

REPLACEMENT VINYL: In the event of wear or damage, pre-sewn replacement vinyl is available from Thuli Tables.